

Isotoner Therapeutic Gloves
Insert for polybag
Die# CWD-05-020Rev1

15.25"

7.75"

7.5"

3.5"

ONE PAIR

FULL FINGER GLOVE



» ISOTONER®
THERAPEUTIC GLOVES

Seamless
SUPPORT™



Wear Anytime
for Comfort

EASES PAIN AND SWELLING
due to Arthritis or Injury

LATEX FREE

UPC

©2007, tates ISOTONER, 9853 International Blvd., Cincinnati, OH 45246
www.isotoner.com
R#22605 CA08966 AR1S115
PINSRTHR2975
ISOTONER and » symbols are registered trademarks of the tates » ISOTONER Corporation.

Eases pain and swelling from:
• Arthritis • Surgery • Repetitive Motion
• And other hand related injuries

CAUTION: If you experience discomfort at any time, discontinue use and consult your physician.
SEE PRODUCT LABEL FOR FABRIC CONTENT AND COUNTRY OF ORIGIN.

FOR MEN AND WOMEN

L M S XS

Position one side of palm at the left, and measure your hand width



All seams designed away from pain sensitive areas like the base of the thumb

Open cuff for easy on, easy off

Lighweight spandex provides even compression throughout the hand to control swelling

Machine Washable

» ISOTONER®
THERAPEUTIC GLOVES

HOW IT WORKS:
ISOTONER Therapeutic Gloves are made of a combination of nylon and spandex material that stretches to provide compression, gently easing the pain of swelling.

USE:
You may use ISOTONER Therapeutic Gloves at anytime day or night when hands are sore or swollen. The massaging properties of the material and gentle compression will provide immediate relief. Always consult your Physician or Therapist for specific instructions related to your care.

TO WEAR:
ISOTONER Therapeutic Gloves are designed with seams on the outside of the glove in order to lessen stress on the hands. Please place gloves on hands keeping the seams on the outside.

The wrist area of the glove contains opening slots in order to make them easier to get on and off. Use this area to pull glove onto the hand.

CONTENTS:
Fiber Content: 80% Nylon, 20% Lycra® Spandex
Latex Free

QUALITY GUARANTEE:
Every ISOTONER Therapeutic Glove is constructed with the finest materials to ensure a consistently high level of quality. If this product does not perform due to defect in materials or workmanship, we will replace it with a similar product of equal value for a period up to one year.

CAUTION:
THIS PRODUCT IS RECOMMENDED FOR THERAPEUTIC USE. PLEASE CONSULT YOUR PHYSICIAN BEFORE USING OR IF SYMPTOMS PERSIST OR WORSEN.

IF YOU EXPERIENCE ANY NUMBING FEELING OR COMPRESSION THAT IS TOO TIGHT AND NOT TOLERATED, DISCONTINUE USE AND ALERT YOUR DOCTOR OR THERAPIST.

CARE INSTRUCTIONS:
Machine wash cold on delicate cycle. Lay flat to dry. Do not place gloves in the dryer or close to other heated areas.

» ISOTONER®
THERAPEUTIC GLOVES

DESCRIPTION		SPOT COLORS		TINTS/SCREENS BUILT COLORS		SPECIAL INSTRUCTIONS:	
ISOTONER - fingerless Therapeutic Insert Card		CYAN	MAGENTA	YELLOW	BLACK	PMS 730	
JOB # CWD-05-31	DATE: 7-31-07	PMS 279		PMS 279		PMS 279	
UPC # PINSRTHR2975	RELEASE DATE:	PMS 279		PMS 279		PMS 279	
DIE # CWD-05-020Rev1	APPLICATION: Illustrator CS2	PMS 279		PMS 279		PMS 279	
FILE NAME: cew0213_fillingtherap2975	THERMAL OUTPUT SIZE: 100%	PMS 279		PMS 279		PMS 279	
PRINTER: CHINA		PMS 279		PMS 279		PMS 279	
FILMS: CHINA	CREATED BY: SAS	PMS 279		PMS 279		PMS 279	
tates » ISOTONER®		PMS 279		PMS 279		PMS 279	