

7.75"

15.25"

7.5"

ONE PAIR OPEN FINGER GLOVES



isotoner® THERAPEUTIC GLOVES

Seamless SUPPORT™

Wear Anytime for Comfort



EASES PAIN AND SWELLING due to Arthritis or Injury

LATEX FREE

**Eases pain and swelling from:**  
 • Arthritis • Surgery • Repetitive Motion  
 • And other hand related injuries

CAUTION: If you experience discomfort at any time, discontinue use and consult your physician.

SEE PRODUCT LABEL FOR FABRIC CONTENT AND COUNTRY OF ORIGIN.

isotoner (UK) Ltd.  
 Eastman House, Radford Crescent,  
 Billerica, Essex CM12 0DN  
 www.isotons.co.uk

RN22605 CA05956 ANS115  
 INS25830CA118R  
 ISOTONER and symbols are registered trademarks of the Isotoner Corporation.

UPC

Machine Washable

Lightweight spandex provides even compression throughout the hand to control swelling

Open fingers for active hands

Open cuff like the base of the thumb for easy on, easy off

All seams designed away from palm sensitive areas

FOR MEN AND WOMEN

**X S M L**

Position one side of palm at the left, and measure your hand width

Seamless SUPPORT™

3.5"

isotoner® THERAPEUTIC GLOVES

**HOW IT WORKS:**

ISOTONER Therapeutic Gloves are made of a combination of nylon and spandex material that stretches to provide compression, gently easing the pain of swelling.

**USE:**

You may use ISOTONER Therapeutic Gloves at anytime day or night when hands are sore or swollen. The massaging properties of the material and gentle compression will provide immediate relief. Always consult your Physician or Therapist for specific instructions related to your care.

**TO WEAR:**

ISOTONER Therapeutic Gloves are designed with seams on the outside of the glove in order to lessen stress on the hands. Please place gloves on hands keeping the seams on the outside.

The wrist area of the glove contains opening slots in order to make them easier to get on and off. Use this area to pull glove onto the hand.

**CONTENTS:**

Fiber Content: 80% Nylon, 20% Lycra® Spandex  
**Latex Free**

**QUALITY GUARANTEE:**

Every ISOTONER Therapeutic Glove is constructed with the finest materials to ensure a consistently high level of quality. If this product does not perform due to defect in materials or workmanship, we will replace it with a similar product of equal value for a period up to one year.

**CAUTION:**

**THIS PRODUCT IS RECOMMENDED FOR THERAPEUTIC USE. PLEASE CONSULT YOUR PHYSICIAN BEFORE USING OR IF SYMPTOMS PERSIST OR WORSEN.**

**IF YOU EXPERIENCE ANY NUMBING FEELING OR COMPRESSION THAT IS TOO TIGHT AND NOT TOLERATED, DISCONTINUE USE AND ALERT YOUR DOCTOR OR THERAPIST.**

**CARE INSTRUCTIONS:**

Machine wash cold on delicate cycle. Lay flat to dry. Do not place gloves in the dryer or close to other heated areas.

isotoner® THERAPEUTIC GLOVES

3.2"