# Bruksanvisning – Catell Active Shoulder

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201001:2

# Instruction for use – Catell Active Shoulder

Catell recommend that the product is fitted and adapted by a licensed or otherwise qualified health professionals. Users with diabetes, sensitive skin, known contact allergies, impaired sensibility or poor circulation must be extra observant and check for pinching, chafing or edema when using the brace. Always check material content below. Contact your prescriber or Catell if questions about the product occur.

## Area of use

The brace supports the users arm and shoulder after injury or surgery.

## Application

1. Open the hook and loop straps on the arm part and on the front part. Place the brace on the shoulder by pulling it up the arm. Picture 1
2. Attach and adjust the chest strap through the front parts plastic buckle. Picture 2
3. Adjust the hook and loop straps on the arm part. Picture 3

When necessary adjust the brace. It is important that the brace fits comfortably without pinching, chafing or affecting blood circulation.

## Material

Polyester, polyamide, elastane. Free from latex.

## Washing instructions

Close the hook and loop straps. Follow the washing instructions below. Use a washing pouch. Let the brace drip dry.

## Size

Size	*Dimensions	Size	*Dimensions	Size	*Dimensions
XS	25 - 27 cm	M	29 - 32 cm	XL	35 - 38 cm
S	27 - 29 cm	L	32 - 35 cm		

\*Measure upper arm circumference. Measurements should be seen as guidelines.

The product is bilateral.

Single patient use.

## Recycling

Separate plastic and textile. Sort according to local regulations.

This device conforms to the requirements of the Medical Device Regulation 2017:745, product class 1.

Manufacturer Catell AB  
Västberga Allé 26, SE-126 30 Hägersten, Sweden | Phone: +46 8 727 95 30 | info@catell.se | www.catell.se

201001:2