**THUASNE**

# CICATREX®

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Scar compression garment

Dear patient,

Your high-quality, made-to-measure CICATREX® compression garment has been delivered to you today. The garment concerned is a custom-made article which meets the essential requirements of directive 93/42/EEC for medical products and is only to be applied by the person named in the patient ID card. So please check if your name is stated correctly in the patient ID card.

You may need to wear this compression garment for a longer period of time, depending on the indication. Please also comply with the instructions given by the physicians, therapists and truss makers. If you experience severe discomfort or un-pleasant sensations you should contact your physician or medical supplies store.

Wearing the CICATREX® compression garment continuously for 24 hours a day helps ensure a positive healing process. Only you personally can make this important contribution.

You have been provided with a high-quality product which has been calculated with the assistance of a computer and manufactured specifically for you. If the compression garment is subjected to modifications not performed by the manufacturer, the guarantee, as well as product liability, will become null and void.

### Attributes

CICATREX® compression garments combine optimal medical treatment with high comfort. They trigger no allergic reactions and can be

readily removed and refitted, thanks to their stretching capability, giving the patient much greater freedom of movement.

### Indications

The CICATREX® compression garment is suitable for patients with the following indications:

- scar hypertrophy and keloid scars (excessive scar tissue)
- scar prophylaxis from thermal, chemical and electrical injuries
- skin transplants
- reconstructual plastic surgery
- severe dermatological lesions
- treatment of lymph oedemas in special cases

### Contraindications

- Advanced peripheral arterial occlusive disease
  - Decompensated heart insufficiency
  - Cardiogenic edema due to heart insufficiency
  - Myocardial infarction
  - Lung edema
- Other (relative) contraindications are:
- Wetting dermatoses
  - Sensitivity disorders
  - Advanced peripheral neuropathy
  - Primary chronic polyarthritis

Thuasne does not assume any liability for failing to observe these contraindications.

## Washing and cleaning directions

To preserve the garment for as long as possible, please care for it regularly observing the following points:

1. Wash it frequently – daily, if necessary – to ensure that all fatty remnants are removed.
2. Use a mild detergent, but without fabric softener.
3. Do not wring out the garment; dry by pressing it between with a towel.
4. Do not expose CICATREX® clothing to direct radiant heat, e.g. from a radiator or by direct sun exposure.

**For reasons of hygiene we recommend a double pack.**

## Subsequent orders

Because of their rapid growth, children and adolescents often have to be supplied with compression clothing more frequently. For this reason care should be taken to ensure an ideal fit, and a new garment should be ordered whenever this is required.

After using a garment for a maximum period of 6 months, all compression parts must be renewed, since the combination of consequent use and frequent washing naturally gives rise to wear and tear, which in turn results in a decrease in compression.

## Composition

### **CICATREX® Nature, knitted:**

54 % viscose, 27 % polyester, 19 % lycra

### **CICATREX® Nature, woven:**

54 % viscose, 37 % polyester, 9 % lycra

### **CICATREX® Rigid:**

81 % polyamide, 19 % lycra

### **CICATREX® Skin:**

59 % polyamide, 41 % DORLASTAN®

### **CICATREX® Silver:**

46 % viscose, 27 % polyester, 19 % lycra, 8 % polyamide, silver-coated

We wish you a positive course of healing and a speedy recovery!

Should you have any further questions, please do not hesitate to contact us, by phone or in writing. We will be delighted to hear from you.

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- IL** THUASNE CERVITEX Ltd. · The Bell House, 20 Hataas St. · Kfar Saba, 44425
- S** THUASNE SCANDINAVIA AB · Box 95 · 131 07 Nacka
THUASNE DEUTSCHLAND GmbH