

# Bruksanvisning – Ringen Fingerfärdig



Catell rekommenderar att produkten provas ut och anpassas av legitimerad eller övrigt utbildad sjukvårdskunnig personal. Brukare med diabetes, känslig hud, kända kontaktallergier, nedsatt känsel eller dålig cirkulation ska vara extra observant för tryck, skav eller ödem under användning. Kontrollera alltid materialinnehåll nedan. Om frågor kring produkten uppstår kontakta din ordinatör, utprovare eller Catell.

## Användningsområde

Fingerring i kort och lång modell som används för korrigering av PIP-leder. Ortosens vinkel går att justera med varmluftspistol.



För utprovning och måttagning behövs Fingerfärdigs storleksset, artikelnummer 5200.



## Material

Polykarbonat alternativt silver. Fri från latex och nickel.

Denna produkt är endast avsedd att användas på en patient.

## Återvinning

Sortera enligt lokala föreskrifter.

Denna produkt överensstämmer med de krav som ställs i den Medicintekniska förordningen 2017:745, klass 1 produkter.



Manufacturer Catell AB  
Västberga Allé 26, SE-126 30 Hägersten, Sweden | Phone: +46 8 727 95 30 | info@catell.se | www.catell.se

201001:1

# Instruction for use – Ringen Fingerfärdig

Catell recommend that the product is fitted and adapted by a licensed or otherwise qualified health professionals. Users with diabetes, sensitive skin, known contact allergies, impaired sensibility or poor circulation must be extra observant and check for pinching, chafing or edema when using the brace. Always check material content below. Contact your prescriber or Catell if questions about the product occur.

## Area of use

Finger ring in short and long model used to correct contractures of PIP-joints. The angle of the orthosis can be adjusted with a hot air gun.



For testing and measuring, you need a ring sizing set, art. number 5200.



## Material

Polycarbonate or silver. Free from latex and nickel.

Single patient use.

## Recycling

Sort according to local regulations.

This device conforms to the requirements of the Medical Device Regulation 2017:745, product class 1.



Manufacturer Catell AB  
Västberga Allé 26, SE-126 30 Hägersten, Sweden | Phone: +46 8 727 95 30 | info@catell.se | www.catell.se

201001:1