Bruksanvisning Fingerstrumpa

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Instruction for use - Fingerstocking

Catell recommend that the product is fitted and adapted by a licensed or otherwise qualified health professionals. Users with diabetes, sensitive skin, known contact allergies, impaired sensibility or poor circulation must be extra observant and check for pinching, chafing or edema when using the stocking. Always check material content below. Any serious incidents occurring relates to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident. Contact your prescriber or Catell if questions about the product occur.

MODEL	COMPRESSION CLASS	ART.NO	SIZE	*DISTAL CIRCUMFERENCE	*PROXIMAL CIRCUMFERENCE
Finger Light	1-2 (15-25mmHg)	7250, 7251	XS	30 mm	50 mm
Finger More	2 (23-32mmHg)	8250, 8251	S	40 mm	60 mm
			M	50 mm	70 mm
			L	60 mm	80 mm
			XL	70 mm	90 mm

*Measurements should be seen as guidelines.

Length of stocking: 10 cm

Properties

- Seams on the outside to minimize pressure on sensitive points.

Use stocking

- In purpose to reduce swelling, as recommended by prescriber.
- With the seam outwards.

Contact your prescriber if

- The swelling increases.
- The stocking causing numbness.
- The stocking causes pain.

Material

Nylon, lycra. Free from latex.

Washing instructions

Follow the washing instructions below. Use a washing pouch. Let dry horizontally.

Single patient use.

Recycling

Sort according to local regulations.

This device conforms to the requirements of the Medical Device Regulation (EU) 2017/745 product class I.