Bruksanvisning - Fingerlift/Thumblift

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Instruction for use – Fingerlift/Thumblift

Catell recommend that the product is fitted and adapted by a licensed or otherwise qualified health professionals. Users with diabetes, sensitive skin, known contact allergies, impaired sensibility or poor circulation must be extra observant and check for pinching, chafing or edema when using the brace. Always check material content below. Any serious incidents occurring relates to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident. Contact your prescriber or Catell if questions about the product occur.

Area of use

The brace support and lift the patients proximal phalanx II to V, or I, when the cause is nerve entrapment, after injury, surgery, chronic pain, osteoarthritis or overload problems. The product is used in combination with Lacy- or Prisma braces.

Application

1. Apply the wrist brace according to its instruction for use.
2. Thread the index finger to the little finger in the straps of Fingerlift, one in each strap, and/or the thumb in Thumblift. Picture 1
3. Attach the Fingerlift, and/or Thumblift on the wrist braces dorsal loop. Picture 2

It is important that the brace fits comfortably without pinching, chafing or affecting blood circulation.

Material

Soft: polyester, viscose, elastane, nylon. Free from latex.

Washing instructions

Close the hook and loop straps and remove the splint, if the brace has one, from the splint pocket. Follow the washing instructions below. Use a washing pouch. Let the brace drip dry.

Size

One size. The product is bilateral.
Single patient use.

Recycling

If the product has a removable splint, take it out and sort according to local regulations.

This device conforms to the requirements of the Medical Device Regulation (EU) 2017/745 product class I.

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Manufacturer Catell AB