## Bruksanvisning – Tvillingförband

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230416:3

## Instruction for use – Tvillingförband

Catell recommend that the product is fitted and adapted by a licensed or otherwise qualified health professionals. Users with diabetes, sensitive skin, known contact allergies, impaired sensibility or poor circulation must be extra observant and check for pinching, chafing or edema when using the brace. Always check material content below. Any serious incidents occurring relates to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident. Contact your prescriber or Catell if questions about the product occur.

### Area of use

The brace supports the users fingers after injury, surgery, chronic pain, osteoarthritis, chronic disease or overuse problems. The brace can also be used for supportive purposes in repetitive movements.

### Application

1. Place your fingers on the loop side of the strap, one on each side of the hooks. Picture 1
2. Place loop around fingers and attach to hook. Remember not to tighten too much. Picture 2
3. Cut off excess straps.

Adjust the hook and loop straps when necessary. It is important that the brace fits comfortably without pinching, chafing or affect blood circulation.

### Material

Nylon. Free from latex.

### Washing instructions

Close the hook and loop straps. Follow the washing instructions below. Use a washing pouch. Let the brace drip dry.

### Size

Available in widths 1,6 cm 2,5 cm 5 cm

The product is bilateral.

Single patient use.

### Recycling

Sort according to local regulations.

This device conforms to the requirements of the Medical Device Regulation (EU) 2017/745 product class I.

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Manufacturer Catell AB