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230416:3

# Instruction for use - Prisma Thumb

Catell recommend that the product is fitted and adapted by a licensed or otherwise qualified health professionals. Users with diabetes, sensitive skin, known contact allergies, impaired sensibility or poor circulation must be extra observant and check for pinching, chafing or edema when using the brace. Always check material content below. Any serious incidents occurring relates to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident. Contact your prescriber or Catell if questions about the product occur.

## Area of use

The brace supports the users thumb, hand, wrist and forearm after injury, surgery, chronic pain, osteoarthritis or overuse problems. The brace can also be used for preventive purposes in heavy or repetitive movements.

## Application

1. The brace has a preshaped volar splint. It can be adjusted if necessary.
2. The splints around the thumb can if needed be adapted to the shape of the users thumb and hand.
3. Open all hook and loop straps. Place your hand in the brace as illustrated.
4. Close all hook and loop straps so that the brace fits comfortably. Adjust the hook and loop straps around the thumb.

Cut off excess hook and loop straps for a cleaner and neater appearance. Adjust the hook and loop straps when necessary. It is important that the brace fits comfortably without pinching, chafing or affecting blood circulation.

## Material

**Stabil Plus:** acrylic, cotton, nylon, polyester. Free from latex.

## Washing instructions

Close the hook and loop straps and remove the splints from the splint pockets. Follow the washing instructions below. Use a washing pouch. Let the brace drip dry.

## Size

Strl	*Dimension	Size	*Dimension	Size	* Dimension
XS	12 - 14 cm	M	16 - 18 cm	XL	20 - 22 cm
S	14 - 16 cm	L	18 - 20 cm		

\*Measure wrist circumference. Measurements should be seen as guidelines. Single patient use.

## Recycling

If the product has a removable splint, take it out and sort according to local regulations.

This device conforms to the requirements of the Medical Device Regulation (EU) 2017/745 product class I.

Manufacturer Catell AB

Örkroken 23, SE-138 40 Älta, Sweden | Phone: +46 8 727 95 30 | info@catell.se | www.catell.se

230416:3