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# Instruction for use – Catell Humerus Lock

Catell recommend that the product is fitted and adapted by a licensed or otherwise qualified health professionals. Users with diabetes, sensitive skin, known contact allergies, impaired sensibility or poor circulation must be extra observant and check for pinching, chafing or edema when using the brace. Always check material content below. Any serious incidents occurring relates to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident. Contact your prescriber or Catell if questions about the product occur.

## Area of use

The brace supports and stabilizes the users arm and shoulder after injury or surgery.

## Application

1. If desired, use the additional sleeve between the arm and the brace. *Picture 1*
2. Safely slide the brace on the arm and place on the shoulder. *Picture 2*
3. Attach the long upper strap around the chest. *Picture 3*
4. Adjust the arm parts straps and closures. Ensure a comfortably and tight fit. *Picture 4*
5. Make sure that all straps are properly secured.

If desired, the plastic parts can be adjusted. When necessary adjust the brace. It is important that the brace fits comfortably without pinching, chafing or affecting blood circulation.

## Material

PE-plastic, polyester, nylon. Free from latex.

## Washing instructions

Detach textile padding parts, close hook and loop straps. Put them in a washing pouch and follow the washing instructions below. The plastic part can be washed with soap and water, no machine wash. Let the brace drip dry.

## Size

SIZE	UPPER ARM CIRCUMFERENCE *
One size, bilateral	24-40 cm

Single patient use.

## Recycling

Separate plastic and textile. Sort according to local regulations.

This device conforms to the requirements of the Medical Device Regulation (EU) 2017/745 product class I.

Manufacturer Catell AB  
Örkroken 23, SE-138 40 Älta, Sweden | +46 (0)8 727 95 30 | info@catell.se | www.catell.se

260218:7