Bruksanvisning – ThumbX

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Instruction for use – ThumbX

Catell recommend that the product is fitted and adapted by a licensed or otherwise qualified health professionals. Users with diabetes, sensitive skin, known contact allergies, impaired sensibility or poor circulation must be extra observant and check for pinching, chafing or edema when using the brace. Always check material content below. Any serious incidents occurring relates to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident. Contact your prescriber or Catell if questions about the product occur.

Area of use

The brace support and stabilizes the users CMC1- and MP-joint of the thumb after injury, surgery, chronic pain, osteoarthritis or overuse problems. The brace can also be used for preventive purposes in heavy or repetitive movements.

Application

1. Adjust the aluminiumbar in the brace to the users thumb and desired position. *Picture 1*
2. Open all hook and loop straps, close the thumb strap. Place your hand in the brace as illustrated in *picture 2*.
3. Close all hook and loop straps so that the brace fits comfortably. Adjust the thumb grip. *Picture 3*

Cut off excess hook and loop straps for a cleaner and neater appearance. Adjust the hook and loop straps when necessary. It is important that the brace fits comfortably without pinching, chafing or affecting blood circulation.

Material

Nylon, polyester, foam, aluminiumbar. Free from latex.

Washing intructions

Close the hook and loop straps. Follow the washing instructions below. Separate wash is recommended. Use a washing pouch. Let the brace drip dry.

Size

SIZE	DIMENSIONS *	SIZE	DIMENSIONS *
S/M	14-17 cm	M/L	17-20 cm

* Measure wrist circumference. Measurements should be seen as guidelines. Bilateral. Single patient use.

Recycling

Sort according to local regulations.

This device conforms to the requirements of the Medical Device Regulation (EU) 2017/745 product class I.

Manufacturer Catell AB
Örkroken 23, SE-138 40 Älta, Sweden | +46 (0)8 727 95 30 | info@catell.se | www.catell.se