



# Dema™ Finger Sleeve

## Indications

To provide even pressure to decrease the edema from the tip to the base of the digit.

## Instructions For Use



1. Measure a piece of the Dema™ Finger Sleeve twice the length of the digit. Trim excess material. Edges will not fray.



2. Place the sleeve on the digit with seam on the outside.



3. Pull the remaining sleeve inside out and over the digit to form a double layer.



4. Pull second layer even with the first layer.

**Note:** Double layers are not always necessary. If a lighter pressure is desired, only one layer can be used over the finger. Simply cut sleeve off at the finger tip edge.

## Sizing

18" Length	55" Length		PIP Circumference	
NC54021	NC54021-55	Petite	1¼" to 1¾"	(3.2 to 4.4 cm)
NC54022	NC54022-55	Small	1⅜" to 2⅛"	(3.5 to 5.4 cm)
NC54023	NC54023-55	Medium	1⅞" to 2⅝"	(4.8 to 6.7 cm)
NC54024	NC54024-55	Large	2⅜" to 2⅞"	(6.0 to 7.3 cm)
NC54025	NC54025-55	X-Large	2¾" to 3"	(7.0 to 7.6 cm)
NC54029	NC54026-55	XX-Large	3" and over	(3.5 and over)