Español

SILOPAD™ Dedil Digital Cap

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WMDE
Bergerweg 18
6085 AT Horn
Holland

www.silipos.com

Made in U.S.A.
Licensed under U.S. Patent number
6,552,109

9565-13

SILOPAD™ Digital Cap

GEL INSIDE

SILIPOS
Innovative Gel Solutions™
www.silipos.com

English

SILOPAD™ Digital Cap

- A soft, comfortable and stretchable fabric fully coated with our proprietary polymer gel (Product#10225 and #10245 are coated with gel at the tip of the cap) which slowly releases a medical grade mineral oil (USP) to soften and moisturize the skin.
- Product is dermatologist tested, washable and reusable.

Indications:

- Helps relieve pressure and friction on the affected digit. Also helps to reduce scar tissue.
- For hammertoes, over/under lapping digits, keratotic lesions, or partially amputated digits. Also may be used for skin or nail problems due to dryness.

Directions for Use:

- Wash and dry either toe or finger.
- Slip product over toe or finger and place the gel directly over the affected area.
- Remove product at least 3 to 4 hours daily to allow skin to breathe.

Recommended Care:

We recommend that you hand wash product. Air dry and lightly dust product with talcum powder after drying.

WARNING: Do not place product on an open wound. If irritation, discomfort or poor circulation occurs, discontinue use and notify your physician *immediately*. KEEP OUT OF REACH OF CHILDREN.

Product #10195	Mini Digital Cap	6 per pkg.
Product #10205	Digital Cap Small/Medium	6 per pkg.
Product #10206	Digital Cap Large/XLarge	6 per pkg.
Product #10235	Mesh Digital Cap	8 per pkg.

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