Español

Half Coated Body Sleevng

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WMDE
Bergerweg 18
6085 AT Horn
Holland

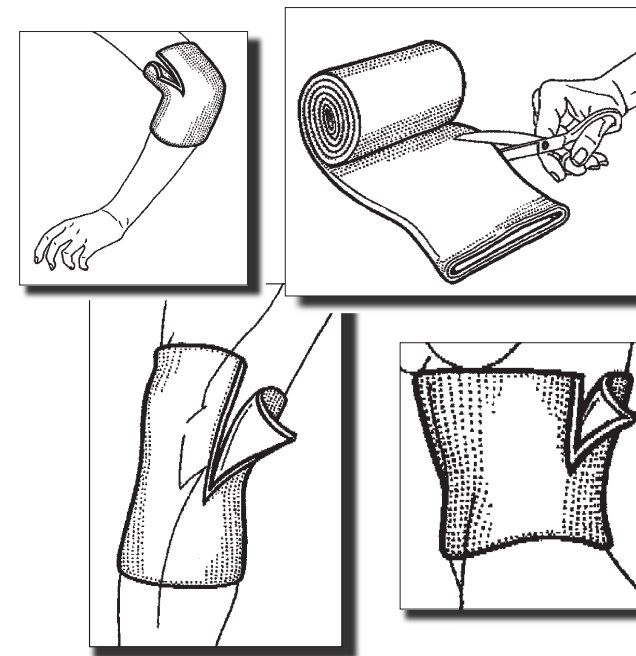
www.silipos.com

Made in U.S.A.
Licensed under U.S. Patent number
6,552,109

Half Coated Body Sleevng

GEL INSIDE

SILIPOS
Innovative Gel Solutions™
www.silipos.com

9578-6

English

Half Coated Body Sleevng

- A nylon/lycra sleeve that is coated on half of it's circumference with Silipos' proprietary gel which gradually dissipates a high grade mineral oil (USP) that moisturizes and protects the skin.
- Product is dermatologist tested, hypo-allergenic, washable, and reusable.

Indications:

- For Donor Sites, Burns, Lateral Epicondylitis, and for use as a protective sleeve against friction, shear forces, and pressure.
- Can be used as Pressure Protector for heels and elbows, as DeQuervain's Pad under splint, as Carpal Pad, or as Ulnar Nerve Sleeve.
- Also used for treating hypertrophic and keloid scars.

Directions for Use:

- Product may be placed on any part of the body. Wash and dry affected area on which product is to be applied.
- Cut product to desired length.
- Roll product on so that gel is in direct contact with the affected areas.
- Remove product at least 3 to 4 hours daily to allow skin to breathe.

Recommended Care:

We recommend that you hand wash product with mild soap and water. Rinse thoroughly. Then, air dry product and lightly dust gel with talcum powder after drying.

WARNING: Do not place product on an open wound. If irritation, discomfort or poor circulation occurs, discontinue use and notify your physician immediately. KEEP OUT OF REACH OF CHILDREN.

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