



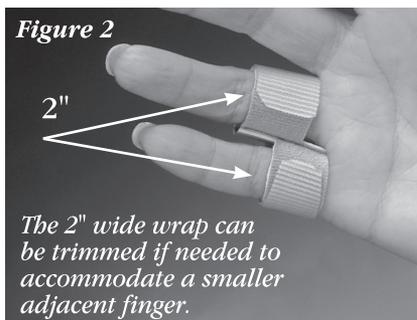

fabriFoam® Finger Buddy Strap

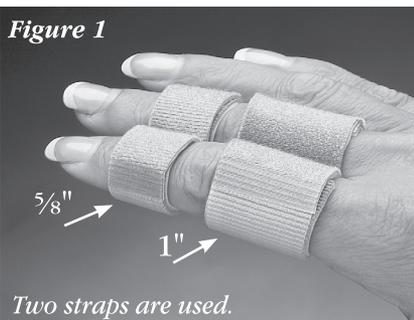

Indications:

Use to hold two fingers together in order to support, protect and/or increase the range of motion of an injured finger caused by such conditions as a ruptured or strained tendon. One or more Buddy Straps may be used on the fingers at the proximal, middle or distal phalanx level. The wrap can also be applied around two or three digits at one time as required for support.

Instructions For Use:

1. Place the strap with the fused section between the two fingers.
2. Bring each long strap around each finger to the fused section. Cut off extra strap length.
3. Secure each strap using the hook tab extensions.

Tip: To apply the straps to different length fingers, cut crosswise and lengthwise as needed. See Figure 2, where a 2" (5.1cm) wide strap was trimmed to fit each finger at a different height.

Care Instructions:

Hand or machine wash with mild soap and allow to air dry.
Do not use a dryer.

fabriFoam® Finger Buddy Strap (Pkg. of 5)

NC15399-1 3/8" (16mm)

NC15399-2 1" (2.5 cm)

NC15399-3 2" (5.1 cm)

To be used under the guidance of a qualified medical professional.

**SINGLE
PATIENT
USE**

