


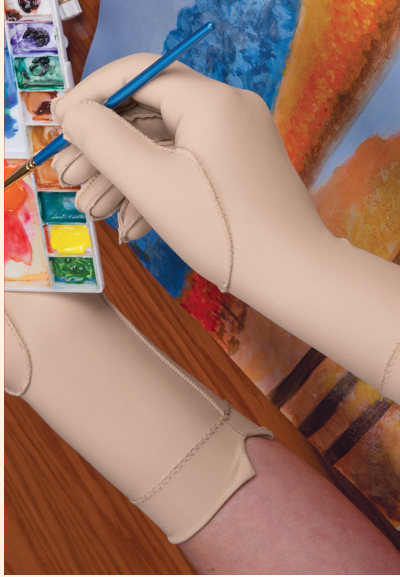

Compression Glove

Soothes and supports swollen or painful joints.

- **Made of an 83% nylon 17% spandex blend.**
- **Outside seams prevent unwanted pressure lines.**
- **Machine washable. Not made with natural rubber latex.**



Care Instructions:

Machine wash, COLD.
Gentle cycle. Lay flat to dry.

Caution: *If compression feels too tight, fingertips change color, or if pain, stiffness or numbness increase, discontinue use.*

Instructions d'entretien :

Lavable en machine à l'eau froide.
Cycle délicat. Faire sécher à plat.
Attention :
Si la compression est trop forte, si le bout des doigts change de couleur, ou si la douleur, la raideur ou l'engourdissement augmente, cesser d'utiliser le produit.

Instrucciones para el cuidado:

Lavar a máquina en frío. Ciclo delicado. Tender al aire sin doblar.
Precaución:
Si la compresión es demasiado fuerte, las yemas de los dedos pierden su color, o si siente más dolor, rigidez o adormecimiento, dejar de usar el producto.

Istruzioni per la cura:

Lavabile a freddo in lavatrice.
Ciclo delicato. Asciugare in piano.

Pflegeanweisungen:

Im Kaltwaschgang waschen.
Schongang. Flach liegend trocknen.

Attenzione:

Interrompere l'uso se la compressione è eccessiva, i polpastrelli cambiano colore, o se dolore, rigidità o intorpidimento aumentano.

Achtung:

Bei zu starker Kompression, Verfärbung der Fingerspitzen oder Zunahme von Schmerzen, Steifigkeit oder Taubheitsgefühl nicht weiter verwenden.

	Size	MP Circumference	Hand
NC53220	X-Small	Less than 7" (Less than 18cm)	Left
NC53221	X-Small	Less than 7" (Less than 18cm)	Right
NC53222	Small	7" to 8" (18 to 20cm)	Left
NC53223	Small	7" to 8" (18 to 20cm)	Right
NC53224	Medium	7-1/2" to 8-1/2" (19 to 22cm)	Left
NC53225	Medium	7-1/2" to 8-1/2" (19 to 22cm)	Right
NC53226	Large	8" to 10" (20 to 25cm)	Left
NC53227	Large	8" to 10" (20 to 25cm)	Right



North Coast Medical, Inc.
www.ncmedical.com



North Coast Medical, Inc.
780 Jarvis Drive, Suite 100
Morgan Hill, CA 95037 – U.S.A.



MedPass SAS

95 bis Boulevard Pereire
75017 Paris – France



REV53220p_0719

Made in China • Fabriqué en Chine • Fabricado en China • Fabbricato in Cina • Hergestellt in China